



POCKET HANDLE




PLEASE READ AND FOLLOW THESE INSTRUCTIONS CAREFULLY

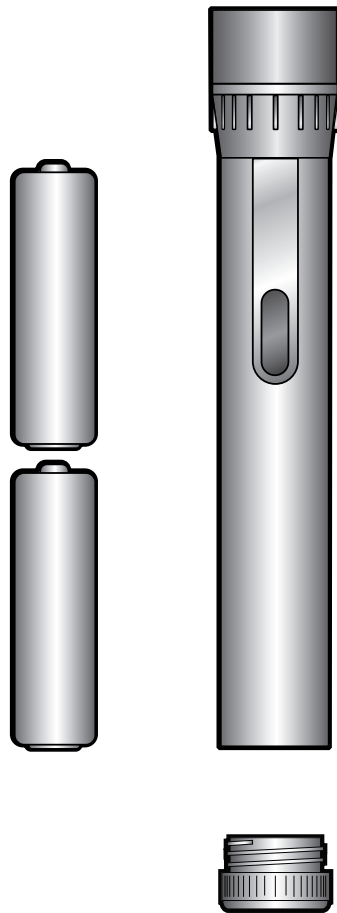

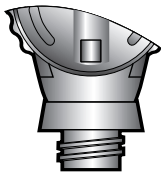


Connection of the instrument heads to the handle.

The connection between the instrument head to the handle is a screw thread. To connect our instrument head connect as shown and rotate in clockwise direction. Ensure the connection between the head and handle is positive.

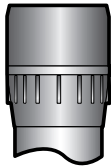

Compatibility

The Keeler Specialist, Professional, Standard and Practitioner Ophthalmoscopes and Keeler Retinoscopes are compatible with Keeler Vista 2.8v and 3.6v Keeler handles.

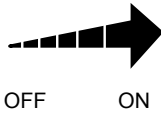

On/Off brightness control

To switch the instrument on, rotate the brightness control as indicated to the right.

To switch off the instrument, rotate the brightness control as indicated to the left.

Inserting/Replacing Batteries

Unscrew battery cap, insert batteries and replace battery cap as shown.

The following dry cell batteries should be used:

Keeler Pocket handle – 2 x 1.5v AA size batteries. – Duracell MN 1500 or equivalent.

Cleaning Instructions

Only manual non-immersion cleaning as described below should be used for the instrument:

1. Wipe the external surface with a clean absorbent, non shedding cloth dampened with a water / detergent solution (2% by volume) or water / isopropyl alcohol solution (70% by volume).
2. Ensure that excess solution does not enter the instrument. Use caution to ensure cloth is not saturated with solution.
3. Surfaces should be carefully hand-dried using a clean non-shedding cloth.
4. Safely dispose of used cleaning materials.

Warranty & Service

No user serviceable parts - all preventative maintenance and servicing must only be performed by authorised Keeler representatives.

Your Keeler handle is guaranteed for 3 years and will be replaced or repaired free of charge subject to the following:

1. Any fault is due to faulty manufacture
2. Proof of purchase accompanies any claim.

MANUFACTURED IN THE UK BY:

Keeler Limited

Clewer Hill Road

Windsor

Berkshire SL4 4AA

England

Tel: +44 (0)1753 857177

Fax: +44 (0)1753 827145

FREEPHONE: 0800 521 251

Distributed by:

Keeler Instruments Inc

456 Parkway

Broomall

PA 19008, USA

Toll Free: 1 800 523 5620

Tel: 610 353 4350

Fax: 610 353 7814

The CE mark on this ☐
product indicates it has ☐
been tested to and ☐
conforms with the ☐
provisions noted within ☐
the 93/42/EEC Medical ☐
Device Directive.



— A —
HALMA GROUP
C O M P A N Y



Distributor