

4Sight Installation Instructions

REV - D



Installation Instructions For Software Update

Please download the update and follow these simple instructions to install the 4Sight update package.

1. **Step One:** Download and copy the installation package to a USB.
2. **Step Two:** Make sure the B-Scan and/or UBM probe is unplugged from the unit.
3. **Step Three:** Power on the 4Sight unit.
4. **Step Four:** Click [Windows OS](#) and enter the password — [malvern19355](#) — to exit desktop.
5. **Step Five:** Open [Control Panel](#) and then select [Programs](#).
6. **Step Six:** Uninstall the 4Sight program.
7. **Step Seven:** Insert the USB disk with the Install Package on it and open the file folder on the disk.
8. **Step Eight:** Right click the package and select [Run as Administrator](#).
9. **Step Nine:** Follow the steps to install the new application.
10. **Step Ten:** Plug the B-Scan probe back into the unit after the installation has finished.
11. **Step Eleven:** Re-boot the 4Sight unit after installation is complete.
12. **Step Twelve:** The most current version should now show on the [Home Screen](#).

For assistance with the device upgrade or any other questions, please feel free to contact Accutome Diagnostic Support at diagnostics@accutome.com or 610.889.0200.



1-800-979-2020 • 610-889-0200 • FAX 610-889-3233 • info@accutome.com • www.accutome.com

Total Solutions Provider for all of your ophthalmic needs.

ACCUTOME SURGICAL

Diamond Knives, Steel Knives, Sutures and Handheld Surgical Instruments.

ACCUTOME CLINICAL

Office Medications and Supplies.

ACCUTOME DIAGNOSTIC

A-Scans, B-Scans, Pachymeters, Tonometer and UBM.

ACCUTOME REPAIR CENTER

- Quick turnaround for most brands of Diamond Knives and Handheld Instruments.
- Diagnostic Equipment.

ORDERING

Call toll free (800) 979.2020 US and Canada
(610) 889.0200 International
+31 (0) 485.350300 Europe
Monday thru Friday 8:30AM - 8PM EST
Fax: (610) 889.3233
Email: info@accutome.com
Web: www.accutome.com

SHIPPING

Ground - Via UPS Ground Service
Air - Via best method

Saturday - Orders must be placed before 3PM EST
Friday to ensure Saturday delivery.

Payment should be made by check, Bank Transfer, Visa,
MasterCard or American Express.

Distributed by: